



3 easy ways to upgrade your ZOLL AED Plus to 2010 Guidelines



Option 1: Do it yourself for free!

- Download the software, <http://www.zoll.com/AEDPlusUpgrade/>
- Beam it to your unit with an infrared USB port
- Keep track of upgrade log and register upgrade with ZOLL
- Place stickers over first two pictures on your unit to represent the upgrade changes.

Option 2: Purchase Software CD, New Stickers and Infrared USB Port directly from ZOLL Medical!

- Part#: 7771-000010-01 for AED Plus 2010 Guidelines Upgrade, Ten Kits (CD and 10 Overlay Label Sets) Price: 100.00 USD
- Part#: 7771-000011-01 AED Plus 2010 Guidelines Upgrade, Single Kit (CD and One Overlay Label Set) Price: 10.00 USD
- Part#: 8000-1200 Infrared USB Port IrDA/USB dongle Price: 19.95 USD (Can also be found online and at office supply stores)

Option 3: Have us do the upgrade for you!

- Located in Portland Metro Area we will come to your location and upgrade your AED for \$100 each (5 or more units \$70.00 each).
- Outside of Portland Metro Area UPS your units to us, we will upgrade and UPS back to you. You pay UPS shipping fees to us we pay return fees. \$100 each (5 or more units \$70.00 each). *International shipping fees additional.*
- We give you a copy of the upgrade log to file for compliance and stickers to place on unit.

WHY YOU WANT TO UPGRADE

- The most important change introduced in 6.24 is the change to deeper compressions. The CPR “bar gauge” and the prompting for PUSH HARDER and GOOD COMPRESSIONS have been changed so that a good compression must now be at least a full 2 inches deep.
- You are not required to do anything other than upgrade your software, but we recommend placing two stickers over the open airway and check breathing pictures because they are no longer used in the 2010 guidelines.
- ZOLL recommends that you register your upgrade with them at: <http://www.zoll.com/medical-products/automated-external-defibrillators/aed-plus/2010-guidelines-upgrade/register/>
- Don't forget to check and document your battery and Padz expiration dates on a monthly basis! Batteries and Adult CPR-D padz are good for 5 years, Pedi and Stat Padz good for 2 years.

Any questions about what's best for you - we can always be reached at 503.269.7996 or Kristi@heartlinkCPR.com – Kristi Lachenmeier, ZOLL AED PLUS Distributor